

The Tech Unit

for PODOLOG NOVA 3s, PODOLOG ECO and PODOLOG ONE



EN

8013602 | Instructions for use for RUCK® MOBIL SYSTEM Tech Unit
REV 08 | Revision status 27/03/2023



- > Diese Gebrauchsanweisung können Sie in weiteren Sprachen unter nachfolgendem Link herunterladen oder telefonisch anfordern.
- > Tämän käyttöohjeen voitte ladata useilla eri kielillä seuraavasta linkistä tai pyytää tietoa puhelimitse.
- > Puede descargar este manual en diferentes idiomas en el siguiente enlace o bien solicitarlo por teléfono.
- > Instrukcje te mogą być pobierane w innych językach pod poniższymi linkami lub telefonicznie
- > Tento návod je také k dispozici v jiných jazycích. Použijte prosím následující odkaz ke stažení návodu nebo si je vyžádejte telefonicky.
- > Šio lietošanas instrukciju citās valodās iespējams lejuplādēt zemāk norādītajā saitē vai pieprasīt telefoniski.
- > This manual is also available in other languages. Please use the following link to download the manual or request it on the phone.
- > Questo manuale è disponibile anche in altre lingue. Lo si può scaricare utilizzando il seguente link o richiederlo per telefono.
- > 此說明書另有提供其他國家語言，請點擊此連結下載檔案，或透過電話詢問下載亦可。
- > Navodila za uporabo lahko v številnih jezikih pridobite na sledeči povezavi ali naročite prek telefona
- > Acest manual este disponibil si in alte versiuni de limba. Accesati link-ul urmator pentru a descarca manualul sau il puteti solicita telefonic.
- > Вы можете загрузить руководства по эксплуатации на других языках кликнув на ссылку или запросив по телефону.
- > Τις οδηγίες χρήσεως μπορείτε να τις βρείτε και σε άλλες γλώσσες, πατώντας στο παρακάτω link για να τις κατεβάσετε ή να τις ζητήσετε τηλεφωνικώς.
- > U kunt de gebruiksaanwijzing in diverse talen downloaden onder de volgende link. Ook kunt u deze telefonisch aanvragen.
- > Ce manuel d'utilisation est disponible aussi en autre langues. Veuillez utiliser le lien suivant pour le télécharger ou demandez-le par téléphone.
- > Bu kullanım kılavuzunu farklı dillerdede aşağıdaki bağlantıdan indirebilir yada telefon ile istiyebilirsiniz.
- > Atsisiškite išmūtą lietuvių kalba paspausdė šią nuorodą, arba paskambinę telefonu.
- > A használati útmutatót magyar nyelven a következő linkről tudja tölteni, vagy kérheti telefonon.
- > ונפסב נתומ שקבל ומ אבה קניכל תופסוב תופשב הערפתה תוארוה תא תירוהל לכות.
- > 여러 언어로 된 본 사용설명서를 다음 링크에서 다운로드하시거나 전화로 요청하실 수 있습니다.
- > Hægt er að sækja þessar notkunarlæðbeiningar á öðrum tungumálum með eftirfarandi tengli eða panta þær símileids
- > Ezen kezelési útmutató további nyelveken a következő linken keresztül tölthető le, vagy telefonon megkérhető
- > この取扱説明書はその他の言語でも以下のリンクでダウンロードしていただくことが可能です。またはお電話でお問い合わせいただければご案内申し上げます。
- > Denne brugsanvisning kan du downloade på flere sprog på efterfølgende link eller bestille telefonisk
- > Denna bruksanvisning på andra språk kan du antingen ladda ned från följande länk eller beställa på telefon
- > Цю інструкцію по експлуатації на інших мовах ви можете завантажити за нижчевказаним посиланням на сайт або запросити по телефону

1. hellmut-ruck.de
2. Search term: "1114001"
3. Downloads

Downloads

> Gebrauchsanweisung / Instructions for use



HELLMUT RUCK GmbH
Daimlerstraße 23 | DE- 75305 Neuenbürg
hellmut-ruck.de

Copyright

© 2023 HELLMUT RUCK GmbH. All rights reserved. This document must not be copied, edited, reproduced or translated without the prior written permission of HELLMUT RUCK GmbH.

Dear customer,

The RUCK® MOBIL SYSTEM Tech Unit is the practical bag for all technical products for professional mobile foot care. The timeless professional exterior of the bag is made of sturdy, water-repellent lorry tarpaulin. The insert with integrated rotatable trays made of shock-proof ABS plastic is hygienic and easy to clean. Unpack – connect – and you're ready! By concentrating on the essentials and dispensing with unnecessary storage space, we have kept the weight of the Tech Unit to a minimum. Thanks to the compact dimensions, it can be conveniently placed wherever it is needed.

The RUCK® MOBIL SYSTEM Tech Unit has space for all PODOLOG foot care drills, the professional magnifying lamp PODOLOG CIRCLE S mobil, the office folder set and a 3-metre extension cable.

Please read the following instructions for use carefully all the way through before using the product. If you have any further questions, please do not hesitate to contact us.

We sincerely thank you for the trust you have placed in our company and extend our best wishes for success in your daily work with your new Tech Unit.

Kind regards, HELLMUT RUCK GmbH

**Convenient storage for
technical equipment**



1 The Tech Unit options

1118401 Bag for PODOLOG NOVA 3/3s
1118501 Bag for PODOLOG ECO/ONE
1114001 Bag backpack for PODOLOG NOVA 3/3s
1114002 Bag backpack for PODOLOG ECO/ONE
1116001 Case backpack for PODOLOG NOVA
1116002 Case backpack for PODOLOG ECO/ONE

**Everything you need
whatever the scenario**



2 The Working Units

1123501 Working Unit S
1118801 Working Unit M

**All the supplies
you need**



3 The Supplies Unit

1118601 Supplies Unit

Table of contents

General information	5
Product description	6
Inserting the foot care drill into the Tech Unit	8
Mounting the magnifying lamp and handpiece holder	14
Cleaning and disinfecting the Tech Unit	15
Warranty	18
Technical specifications	19



Easy to pack

Video at
hellmut-ruck.de



Declaration relating to these instructions for use

HELLMUT RUCK GmbH reserves all rights to modify and supplement these instructions for use. HELLMUT RUCK GmbH has taken great care to keep these instructions free of inaccuracies and omissions. Please contact us if you find any errors or inaccuracies. However, HELLMUT RUCK GmbH assumes no liability for mistakes in these instructions for use or consequential damage resulting from the provision of these instructions for use or their application when using the product.

These instructions for use must not be disseminated or supplemented without the express approval of HELLMUT RUCK GmbH.

Symbols used



This symbol identifies a potential point of danger for people or for the product itself. It must be taken note of.



This symbol identifies useful information. Additional information about the product and how to use it is provided here.



Packaging

The packaging has been designed to protect the product while in transit. The packaging materials used have been selected with due regard for environmental issues and can be recycled.

Convenient storage for technical equipment



1 The Tech Unit options

1118401 Bag for PODOLOG NOVA 3/3s
1118501 Bag for PODOLOG ECO/ONE
1114001 Bag backpack for PODOLOG NOVA 3/3s
1114002 Bag backpack for PODOLOG ECO/ONE
1116001 Case backpack for PODOLOG NOVA
1116002 Case backpack for PODOLOG ECO/ONE

Everything you need whatever the scenario



2 The Working Units


1123501 Working Unit S
1118801 Working Unit M

All the supplies you need



3 The Supplies Unit

1118601 Supplies Unit

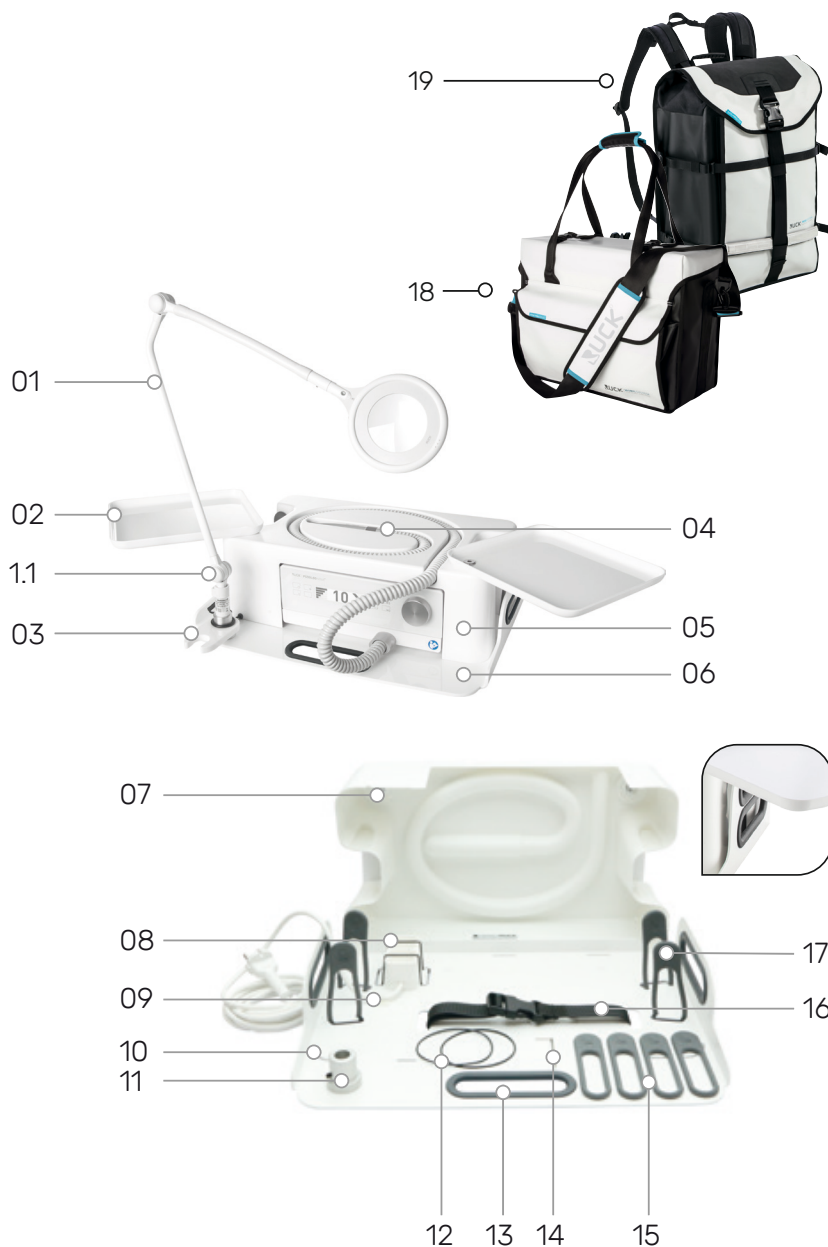

- 01 PODOLOG CIRCLE S mobil Magnifying Lamp (optional)
- 1.1 Joint cap
- 02 Rotatable instrument trays
- 03 Rotatable handpiece holder
- 04 Handpiece storage location
- 05 Drill housing
- 06 Base plate


- 07 Drill housing
- 08 Multi-socket outlet secured with O-ring
- 09 Cable fixing
- 10 Guide groove for drill housing
- 11 Lamp bracket with headless screw
- 12 O-ring (1 x for the power supply of the PODOLOG CIRCLE S, 1 x spare)
- 13 Silicone grip insert for handle
- 14 Allen key for lamp brackets
- 15 4 x Silicone strap (spares)
- 16 Fixation strap for foot care drill
- 17 Silicone straps for drill housing

- 18 Tech Unit Bag
- 19 Tech Unit Bag backpack

Inserting the foot care drill into the Tech Unit



1. Release silicone straps.

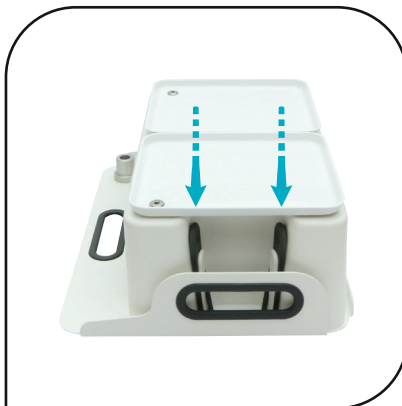

2. Release silicone straps.



3. Release silicone straps.



4. Remove drill housing.



5. Base plate with four recesses for the drill feet.



6. Place the foot care drill directly into the four recesses for the drill feet.



7. Foot care drill rests properly in its position.



8. Strap the foot care drill to the base plate with the fixation strap. If necessary, re-open the fixation strap and tighten it.



9. Plug power cable of the foot care drill into the multi-socket outlet.



10. Stow the power cable to the left of the foot care drill.




11. Insert the black cable bushing into the front recess of the drill housing.



12. Interlock the silicone straps into each other.




13. Insert the drill housing into the guide grooves of the base plate.

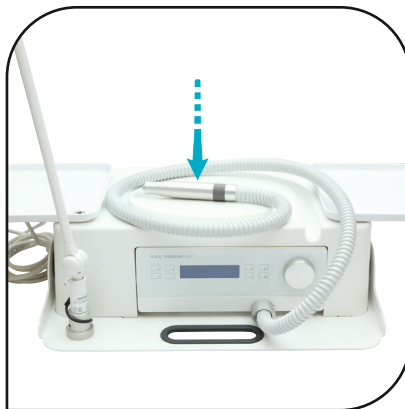


14. Align the suction hose of the foot care drill.



15. Insert the handpiece into the handpiece storage location.

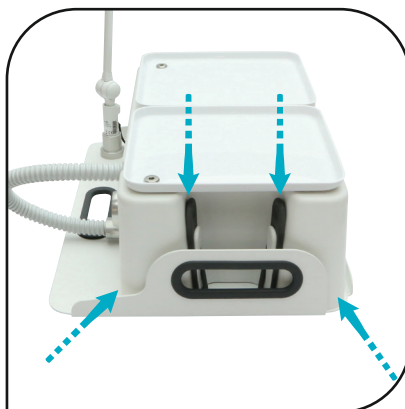

16. On the right side, connect and secure the silicone straps to the drill housing. Secure the drill housing into the guide grooves of the base plate.

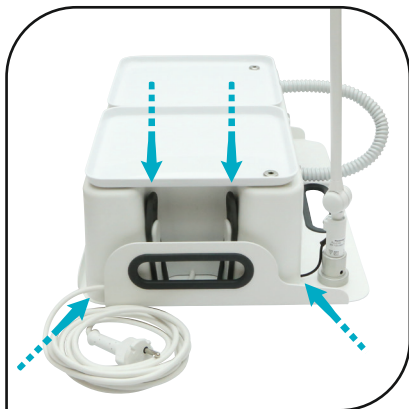

17. On the left side, connect and secure the silicone straps to the drill housing. Secure the drill housing into the guide grooves of the base plate.



18. Roll up the extension cable of the multi-socket outlet and place it in the designated space.



19. Move the magnifying lamp through the suction hose.

Mounting the magnifying lamp and handpiece holder



1. Remove the drill housing.



2. Unscrew the headless screw from the lamp bracket.



3. Put the handpiece holder and then the O-ring over the lamp base.



4. Tighten the headless screw of the lamp bracket. The lamp should still be able to turn easily.



5. Use the O-ring to secure the power cable (use two if necessary).



6. Secure the power cable on the power adapter and plug it into the multi-socket outlet.

Cleaning, disinfecting and caring for the Tech Unit



Always unplug the mains plug before cleaning and/or disinfecting.



Do not spray cleaning and disinfecting agents into the multi-socket outlet.



Cleaning and disinfection

Disinfect the contact surfaces after each treatment with RUCK® alcohol-free surface disinfection wipes, (item no. 2967002, item no. 2967102 or item no. 2967202) for medical devices. Please observe the exposure times of the disinfectants according to the product labelling.

Danger of damage due to improper care of surfaces. Do not use any concentrated disinfectants, solvents, cleaning agents or liquids that contain alcohol (> 50 % alcohol content). Do not use spray disinfectant.

Take great care to ensure that the cleaning utensils used are clean. If mineral contaminants such as grains of sand or dust are embedded in the cleaning agents, this can destroy the surface. Do not use any scouring agents, such as abrasive cleaners, cleaning sponges or similar items that may contain abrasive particles. Residues of leaked fluids must be removed from the surface immediately with a soft cloth.

For gentle cleaning of the surfaces, use a soft cotton cloth or the RUCK® microfibre cloth which is gentle on surfaces (item no. 29616). Caution: Other cleaning cloths with plastic fibres can have an abrasive effect and consequently they can damage the surface.

Over time, traces of use can appear on the surfaces. These are more or less noticeable depending on the lighting. They do not constitute a loss of function. As is the case with all surfaces over the long term, due to the wear involved in the daily routine of the medical practice, they cannot be avoided. Spots or stains caused by liquids that may corrode the surface (such as propolis, tea tree oil, cleaners that contain chlorine or similar substances) are categorically excluded from the warranty.



Danger due to disease transmission!

Insufficient disinfection can lead to transmission of disease. Always comply with the requirements of the responsible national body for hygiene and disinfection. Use the RUCK® alcohol-free surface disinfection wipes (item no. 2967002, item no. 2967102 or item no. 2967202). Do not use any spray disinfectants. It must be possible to effectively evaporate surface disinfectants before the next treatment; allow surface disinfectants to thoroughly dry as specified by the manufacturer, but do not wipe surfaces dry.



1. Remove the Tech Unit from the bag.



2. Remove the drill housing.



3. Remove all components from the base plate.



4. The Tech Unit can now be completely cleaned. In addition, the silicone straps may be removed for cleaning.



5. Lubricate the magnifying lamp socket in the light bracket regularly with RUCK® maintenance spray (item no. 2969001).



6. Lubricate the locking nuts on the rotatable instrument trays regularly with RUCK® maintenance spray (item no. 2969001).



7. Readjust lamp hinges. For this, remove joint cap and secure with open end wrench (included with all lamps).

Warranty

The product comes with a 2-year warranty.

Normal wear and tear of surfaces, handles, straps and/or moving components such as rollers, bearings, plug connections, suction hoses, and so on, are excluded from this warranty. Traces of use and signs of wear and tear due to age, service life and operation which are based on normal use corresponding to the performance specifications and/or the nature of the product are in accordance with the contract and are not defects from a legal standpoint.

The warranty is valid from the date of purchase when the product was first bought. The date of purchase must be confirmed by the sales receipt. Within the scope of the warranty, HELLMUT RUCK GmbH will, at no charge, either repair or replace the defective products, at RUCK's discretion. This requires carriage-paid shipment of the defective product together with the purchase receipt. The local agent is responsible for claims made outside of Germany. In case of doubt, please check with RUCK before returning the product.

The warranty becomes null and void if we find that the problem was caused by improper or incorrect installation, assembly or usage, by non-observance of the instructions for use, by external intervention or as a result of an unauthorised repair or modification. The warranty is limited to repair or replacement of the product. Further liability (loss of revenue or earnings in particular) is excluded. However, if it is determined that it is a problem that is not covered by the terms of the warranty or that the period of the guarantee has elapsed, any costs for inspection and repair must be borne by the customer.

We reserve the right to make changes to and deviations from the technical design.

Technical specifications

RUCK® MOBIL SYSTEM Tech Unit Bag

Dimensions W/D/H: 47/18/35 cm

1118401 Bag for PODOLOG NOVA 3/3s

1118501 Bag for PODOLOG ECO/ONE

RUCK® MOBIL SYSTEM Tech Unit Bag backpack

Dimensions W/D/H: 35/22/48 cm

1114001 Bag backpack for PODOLOG NOVA 3/3s

1114002 Bag backpack for PODOLOG ECO/ONE

RUCK® MOBIL SYSTEM Tech Unit Case backpack

Dimensions W/D/H: 39/26/46 cm

1116001 Case backpack for PODOLOG NOVA

1116002 Case backpack for PODOLOG ECO/ONE





HELLMUT RUCK GmbH
Daimlerstraße 23
DE -75305 Neuenbürg
fon +49 (0)7082. 944 20
fax +49 (0)7082. 944 22 22
kontakt@hellmut-ruck.de

HELLMUT RUCK LTD
Raceside Barn, Kirksanton
Millom, GB-LA18 4NW
hellmut-ruck.de

RUCK®