

Paraffin Heater

Instructions for Use



Item no. 3020001

HELLMUT RUCK Paraffin Heater

- » Diese Gebrauchsanweisung können Sie in weiteren Sprachen unter nachfolgendem Link herunterladen oder telefonisch anfordern.
- » Tämän käyttöohjeen voitte ladata useilla eri kielillä seuraavasta linkistä tai pyytää tietoa puhelimitse.
- » Puede descargar este manual en diferentes idiomas en el siguiente enlace o bien solicitarlo por teléfono.
- » Instrukcje te mogą być pobierane w innych językach pod poniższym linkiem lub telefonicznie
- » Tento návod je také k dispozici v jiných jazycích. Použijte prosím následující odkaz ke stažení návodu nebo si je vyžádejte telefonicky.
- » Šo lietošanas instrukciju citās valodās iespējams lejuplādēt zemāk norādītajā saitē vai pieprasīt telefoniski.
- » This manual is also available in other languages. Please use the following link to download the manual or request it on the phone.
- » Questo manuale è disponibile anche in altre lingua. Lo si può scaricare utilizzando il seguente link o richiederlo per telefono.
- » 此說明書另有提供其他國家語言，請點擊此連結下載檔案，或透過電話詢問下載亦可。
- » Navodila za uporabo lahko v številnih jezikih pridobite na sledeči povezavi ali naročite prek telefona
- » Acest manual este disponibil si in alte versiuni de limba. Accesati link-ul urmator pentru a descarca manualul sau il puteti solicita telefonic.
- » Ви можете загрузить руководство по эксплуатации на других языках кликнув на ссылку или запросив по телефону.
- » Τis οδηγίες χρήσεως μπορείτε να τις βρείτε και σε άλλες γλώσσες, πατώντας στο παρακάτω link για να τις κατεβάσετε ή να τις ζητήσετε τηλεφωνικώς.
- » U kunt de gebruiksaanwijzing in diverse talen downloaden onder de volgende link. Ook kunt u deze telefonisch aanvragen.
- » Ce manuel d'utilisation est disponible aussi en autre langues. Veuillez utiliser le lien suivant pour le télécharger ou demandez-le par téléphone.
- » Bu kullanım kılavuzunu farklı dillerdede aşağıdaki bağlantıdan indirebilir yada telefon ile istiyebilirsiniz.
- » Atsisiųskite instrukcija lietuvių kalbą paspaudę šią nuorodą, arba paskambinę telefonu.
- » A használati útmutatót magyar nyelven a következő linkről tudja tölteni, vagy kérheti telefonon.
- » פולטב ותוא שקבל או אבה קנילב תופסון תושב חלעפה תוארוו תא דירוחל לכות.
- » 여러 언어로 된 본 사용설명서를 다음 링크에서 다운로드하시거나 전화로 요청하실 수 있습니다.
- » Hægt er að sækja þessar notkunarléiðbeiningar á öðrum tungumálum með eftirfarandi tengli eða panta þær símileidis
- » Ezen kezelési útmutató további nyelveken a következő linken keresztül tölthető le, vagy telefonon megkérhető
- » この取扱説明書はその他の言語でも以下のリンクでダウンロードしていただくことが可能です。またはお電話でお問い合わせいただければご案内申し上げます。
- » Denne brugsanvisning kan du downloade på flere sprog på efterfølgende link eller bestille telefonisk
- » Denna bruksanvisning på andra språk kan du antingen ladda ned från följande länk eller beställa på telefon
- » Цю інструкцію по експлуатації на інших мовах ви можете завантажити за нижчеказаним посиланням на сайті або запросити по телефону



1. hellmut-ruck.de
2. Search term: "3020001"
3. Downloads

Downloads

> [Instruction for Use](#)



**XANITALIA s.r.l. via Divisione Acqui 53
61122 Villa Fastiggi – Pesaro – Italy**

Distribution by:

HELLMUT RUCK GmbH | DE - 75305 Neuenbürg | hellmut-ruck.de

Copyright

© 2023 HELLMUT RUCK GmbH. All rights reserved. This document must not be copied, edited, reproduced or translated without the prior, written permission of HELLMUT RUCK GmbH.

Declaration relating to these instructions for use

HELLMUT RUCK GmbH reserves all rights to modify and supplement these instructions for use. HELLMUT RUCK GmbH has taken great care to keep these instructions free of inaccuracies and omissions. Please contact us if you find any errors or inaccuracies. However, HELLMUT RUCK GmbH assumes no liability for mistakes in these instructions for use or consequential damage resulting from the provision of these instructions for use or their application when using the product.

These instructions for use must not be disseminated or supplemented without the express approval of HELLMUT RUCK GmbH.

The manufacturer cannot be made responsible for damage that is caused by use that deviates from the intended use or failure to comply with the safety instructions and warnings.

The instructions for use are intended for persons who carry out paraffin treatments professionally. The instructions for use are an integral part of the product. They contain important information on the commissioning, use and correct maintenance of the product. Please read these instructions for use carefully before commissioning and observe the information in order to operate the device correctly and safely. Keep the instructions for use in a safe place.

Kind regards, HELLMUT RUCK GmbH

Contents

Symbols used..... 5

Intended use..... 6

 Notes on treatment..... 6

Basic safety instructions..... 7

 Hazards due to electricity..... 7

 Hazards due to damaged device..... 8

 Hazards due to unauthorised operation..... 8

 Hazards due to wetness and humidity 8

 Hazards due to incorrect use 9

Product description..... 10

Operation..... 11

Cleaning..... 12

 Cleaning the empty device 12

 Removal of old wax..... 12

Disposal 13

Directives 13

Technical specifications 13

Symbols used



Danger to persons. Observe the note marked with this symbol to avoid personal injury.



Endangering the condition or functionality of the product or endangering the environment.



Manufacturer



CE conformity marking



Note that the product must be taken to a separate collection point for reprocessing and recycling.



Device with protection class II (protective insulation). Does not require earthing, as protection is provided by the double insulation.



The device must not be used near bathtubs, sinks or other containers with water.

Intended use

The device is intended exclusively for heating and melting paraffin wax with a low melting point.

The device is intended for professional use only.

The device may only be operated indoors.

Any other use is non-intended use.

The manufacturer and supplier accept no liability for damage caused by improper use.

Notes on treatment

The treatment of persons with melted paraffin is the responsibility of the person treating them.

Please note that this treatment is not suitable for persons with venous insufficiency (varicose veins), weak capillaries, spider veins or skin irritations.

The temperature of the liquid wax is adjustable.

Maximum temperature: 58°C (136.4°F).

Make sure that the wax is not too hot.

Only use the wax of a paraffin filling for one person.

Basic safety instructions

Hazards due to electricity

**WARNING**

Only connect the device to a suitable mains supply. Observe the information on the type plate. Incorrect operating voltage can lead to hazards due to electrically overloaded components.

Do not connect the device to the power supply until the following conditions are met:

- › The mains supply to which the device is connected must comply with the applicable regulations for electrical installations in residential buildings.
- › The mains supply is designed for the maximum power of the device.
- › The socket and plug of the device match.

If in doubt, consult a qualified electrician.

For better protection, we recommend installing a residual-current circuit breaker in the circuit supplying power to the device. Maximum nominal operating residual current: 30 mA. Consult a qualified electrician for this.

Observe the basic rules for handling electrical equipment:

- › If possible, do not use adapters, multiple sockets or extension cords. If the use of these elements is necessary after all, only use adapters that comply with the applicable safety standards and that are designed for the maximum power of the device.
- › Do not pull on the mains cable or on the device to pull the plug out of the socket.
- › Disconnect the device from the mains supply before cleaning and maintenance work.

Hazards due to damaged device



WARNING

Only use the device if it is undamaged. Unplug the mains plug if you notice any unusual heat development, smell or smoke.

Do not use the device if it has been subjected to a strong impact, for example if the device has fallen on the floor. Even if no external damage is visible, electrical components may be damaged. There is then a risk of electric shock when touching. In this case, have the device checked by the manufacturer or an authorised service centre before using it again.

Hazards due to unauthorised operation



WARNING

Do not leave the device unattended when it is switched on.

Ensure that only authorised persons work with the device. Ensure that the device cannot be used by children or persons with disabilities.

Also keep the packaging material away from these people. Parts of the packaging material may be a suffocation hazard.

Hazards due to wetness and humidity



WARNING

Always keep the device dry. Do not expose the device to the elements.

Do not touch the device with wet or damp hands or when you are barefoot.

Do not operate the device near bathtubs, sinks or other containers with water.

Do not hold the device under running water or immerse it in water or any other liquid.

Do not touch the device if it has fallen into a liquid.

Danger of electric shock!

Always disconnect the device from the mains after use, as proximity to water can be dangerous even when the device is switched off.

Hazards due to incorrect use



DANGER

Do not operate the device in rooms containing flammable or explosive substances or vapours. These include alcohol, petrol, insecticides and sprays. Keep combustible materials such as paper, cardboard, textiles and cloths away from the device.

Danger of fire or explosion!

Always disconnect the device from the mains supply when you are not working with the device.

Do not wrap the mains cable around the paraffin container. This could damage the cable and cause a hazard.



WARNING

Remove the paraffin wax packaging before filling the wax.

Only fill wax in a solid state.

Never fill more than 3 kg of wax.

The device must not be empty when switched on. Only switch on the device when there is wax in the device.

Do not fill the device with water. Do not put any liquids or substances other than paraffin into the device.

Keep the room where the paraffin wax is stored clean.




WARNING

Wax that is too hot on the skin can cause pain. Unpredictable, jerky movements of the person being treated can lead to further hazards.

Only dip hands, elbows or feet into liquid wax when the temperature control knob is set to a value lower than 5 and the red indicator light is NOT lit.

Before carrying out any cleaning work: Switch off the device, disconnect the mains cable and wait until the unit has cooled down completely.

Product description

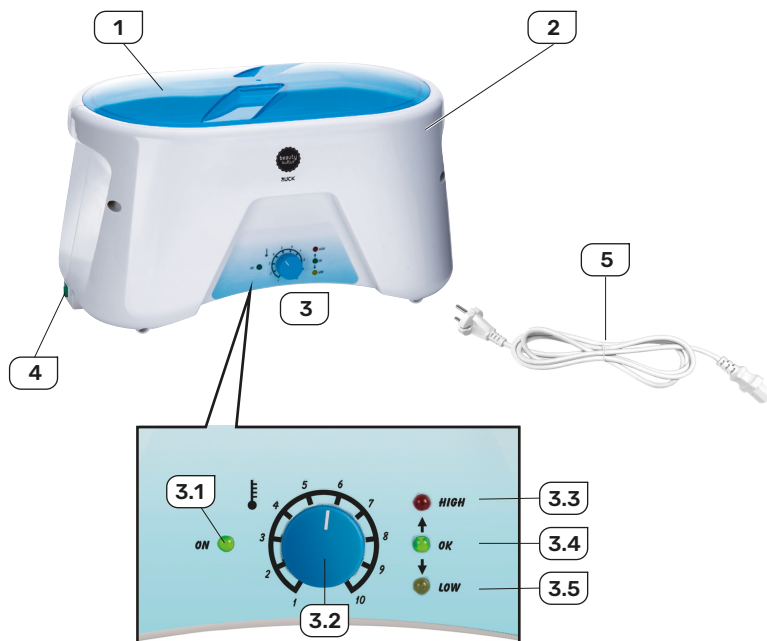

- 1 Lid
- 2 Paraffin container
- 3 Control unit
 - 3.1 Power indicator (ON)
 - 3.2 Temperature controller
 - 3.3 Overtemperature indicator light (HIGH)
 - 3.4 Operating temperature indicator light (OK)
 - 3.5 Low temperature indicator light (LOW)
- 4 Mains switch
- 5 Mains cable with plug CEE 7/17

Operation

1. Place the device on a stable and level surface.
2. Make sure that the paraffin container is dry. There must be no more cleaning liquid on the walls of the container.
3. Connect the device to a socket using the mains cable.
4. Fill 3 kg of paraffin wax into the paraffin container in such a way that the wax has surface contact with the bottom of the container.
5. Close the lid.
6. Turn on the mains switch.
7. Turn the temperature control knob to 10 (maximum temperature).



The device starts to heat.

The LOW indicator light is on as long as the desired temperature has not yet been reached.

When the desired temperature is reached, the OK indicator light lights up.

8. After 150 minutes, turn the temperature control knob to a value between 3 and 5.
9. Make sure that the red HIGH indicator light has gone out. Wait until this is the case, if necessary.



The melted wax is at the correct application temperature when the temperature control knob is set to a value lower than 5 and the OK indicator light is on.



Cleaning

Cleaning the empty device

**WARNING**

Before carrying out any cleaning work: Switch off the device, disconnect the mains cable and wait until the unit has cooled down completely.



Do not hold the device under running water or immerse it in water or any other liquid.

Use a soft cloth soaked in cleaning fluid to remove wax residues on the housing. Use RUCK® beautykultur Equipment Cleanser, item no. 4080701, to gently clean the device.

Do not use corrosive or abrasive cleaning agents or solvents to clean the device. Use only the recommended cleaning fluid. Other cleaning agents may damage the device or produce toxic fumes when heated.

After cleaning, wait until the cleaning liquid has completely evaporated before switching the device on again.

Removal of old wax

If the paraffin is solid, switch on the device for a few minutes until the edges of the paraffin have melted and come away from the wall of the container.
Switch the device off and unplug the mains plug.

Press the paraffin block down slightly at one end, causing it to tilt. Now you can take the paraffin out of the container. Do not use sharp or pointed objects to remove hard paraffin.

Disposal



Ensure proper disposal

Incorrect disposal endangers our environment



The device falls under EU Directive 2012/19/EU. The device must not be disposed of with household waste in countries of the European Union.

Please use available return and collection systems for disposing of the device at the end of its service life.

The product packaging is made of recyclable materials. Separate the individual materials and dispose of them in accordance with the applicable environmental protection laws.

Directives

The device complies with the following directives:

- › Low Voltage Directive 2014/35/EU
- › EMC Directive 2014/30/EU
- › RoHS Directive 2011/65/EU

Technical specifications

- › Supply voltage: 220-230V~ 50/60Hz
- › Electrical power: 150 W
- › Weight: 2.2 kg
- › Filling quantity: 3.0 kg
- › Dimensions: 380 x 220 x 215 mm

This image shows a single sheet of white paper with horizontal blue ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

[illegible]



HELLMUT RUCK GmbH
Daimlerstr. 23
75305 Neuenbürg, Germany
Tel. +49 (0)7082 944 20
Fax +49 (0)7082 944 22 22
E-mail: kontakt@hellmut-ruck.de
Website: hellmut-ruck.de