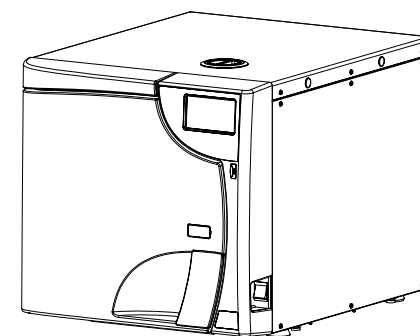
SciCan BRAVO™ G4

Chamber Autoclave

Guida Rapida Utente
User Quick Start Guide
Guide rapide utilisateur
Benutzer-Kurzanleitung
Guía rápida Usuario
Szybki Przewodnik Użytkownika
Kullanıcı Hızlı Ba_latma
Hurtigstart For Bruker Rychlé
Uživatelský návod
Vlugge Start
Γρήγορος Οδηγός Χρήστη
Инструкции Для Быстрого Применения
Guia Rápido do Utilizador
Snabbstart för användare
Käyttäjän pikaopas
للمستخدم السريع الدليل
Gyors használati útmutató
Кратко ръководство на потребителя
Kvikguide
Trumposios instrukcijos
Ghid de utilizare rapida
Uživatelská příručka
快速使用指南
Короткий посібник користувача
Kiirejuhend kiireks käivitamiseks
Brzi vodič za korisnike




Manufactured by: Cefla S.C., Via Selice Provinciale, 23/A, 40026 Imola (BO), Italy
T +39 0542 653111
F +39 0542 653344

REF. 97055191
Rev.01 (2020-04)

SD-590 EU ML R1. BRAVO is a trademark of SciCan Ltd.
A Coltene Group company.

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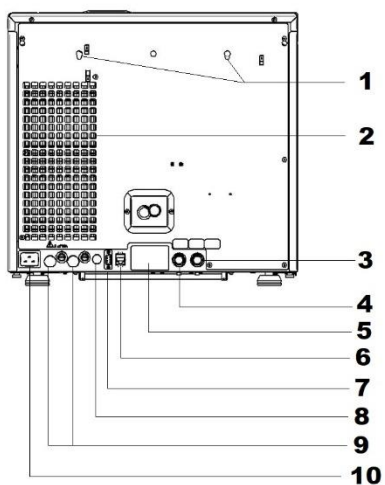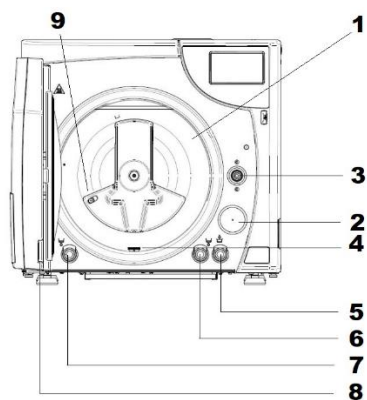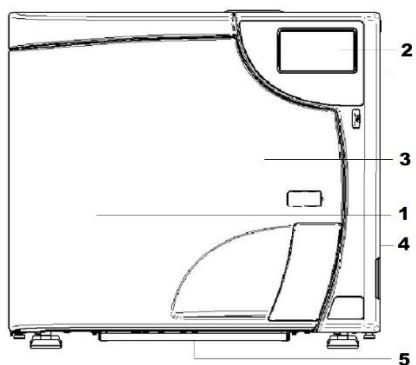NOTE

COMPLETE USER MANUAL AVAILABLE ON SUPPLIED USB DEVICE.

Rif.	Description
1.	Model
2.	LCD touch screen control panel
3.	Door
4.	Power switch
5.	Dust filter

Rif.	Description
1.	Sterilisation chamber
2.	Bacteriological filter
3.	Door locking system
4.	Water drainage filter
5.	Demineralised / Distilled water top-up quick coupling
6.	Water reservoir drainage quick coupling
7.	Waste water drainage quick connector
8.	Door
9.	Steam diffuser

Rif.	Description
1.	Fastening slots for rear spacers
2.	Heat exchanger
3.	Connection for direct water drainage
4.	Connection for demineralised / distilled water automatic filling (only for PURE 100 / 500 and automatic filling accessory kit)
5.	Data plate
6.	Ethernet cable connection (max. length 29 m)
7.	Serial cable connection
8.	Automatic filling electrical connection (only for PURE 100 / 500 and automatic filling accessory kit)
9.	Network fuses
10.	Power cable connection

CYCLES	TYPE OF LOAD	MAX KG		
		17	22	28
B 134°C Universal	Unwrapped porous materials	1,00	1,25	1,50
	Porous materials in single package	0,75	1,00	1,25
	Porous materials in double package	0,60	0,75	0,90
B 121°C Universal	Solid and hollow materials in single package	3,00	4,00	5,00
B 134°C Prion (*)	Solid and hollow instruments in double package	1,50	2,00	2,50
S 134°C Hollow	Unwrapped hollow instruments	6,00	7,50	9,00
	Unwrapped solid instruments	6,00	7,50	9,00
S 134°C Solid	Unwrapped solid instruments	6,00	7,50	9,00
	Solid instruments in single package	3,00	4,00	5,00

(*) NOT AVAILABLE FOR US MARKET

NOTE

Definition of hollow loads in accordance with standard EN 13060:2014 + A1:2018

In this manual the term "hollow loads" refers both to the elements defined as "narrow lumen" (paragraph 3.18 EN 13060:2014 + A1:2018) and to the elements defined as "simple hollow" (paragraph 3.30 EN 13060:2014 + A1:2018).

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