# Hochleistungs LED Lampe

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Greppmayr GmbH Hainbuchenring 4 D-82061 Neuried

# High Performance LED Lamp

#### FOR HARDENING OF UNGUISAN BLUE LIGHT GEL

These instructions for use contain important information to get in touch with the functions of the lamp. Please keep this leaflet save to check up anytime.

## PRODUCT ATTRIBUTES:

- Splash water proof aluminium case
- 3 Watt high power –LED
- Wave lenght: Bluelight 455 465 nm

#### INTENDED USE:

Only for hardening (polymerisation) of Unguisan Blue Light Gel in practice or laboratory.

#### **NON-INTENDED USE:**

- · Hardening of other light curing materials
- Illumination

#### SAFETY ADVICE:

- Never open the gadget apart from battery case at the end of the lamp.
- Do not unscrew the lamp head!
- We recommend to wear suitable protective glasses when using the lamp.

Electromagnetic compatibility: Concerning the electromagnetic compatibility the lamp corresponds to the demands of the directive 2014/30/EU.

#### Attention:

Photobiological security (ocular security): The lamp is classified according to the norm IEC 62471 in the risk group 2. The danger of damaging the net skin by optical radiation (blue light 400 nm to 780 nm) exists. Measured data fall below critical values.

#### Warning:

<u>Danger of glare:</u> The strong glare of the lamp can cause temporarily limited eyesight, which can lead to irritations. interferences or accidents.

#### Therefore:

- Never look directly in the beam of light. Never focus the beam of light into other people's eyes
- Do not direct beam of light upon person or animals, because they can be blinded.
- . Do not use optical instruments for examination

of the beam of light.

- Do not use several lamps at the same time, because this raises the optical radiation and therefore the danger of net skin damage.
- Instruct users concerning the right handling.
  To Do:

If a beam of light meets the eyes, close eyes and move the head from the beam of light.

We are not liable for damage of people or property, caused by wrong use! In that case the warranty will be cancelled immediately.

#### HANDLING:

- Push the button at the lamp end to switch light on / off.
- Shifting the lamp head which focus the light beam.

#### APPLICATION:

- Only use lamp for professional harding of Unquisan Blue Light Gel.
- Distance of exposition: 1 3 cm. Do not touch gel with lens. If it happens, turn off the lamp and wipe off gel with a dry fabric.
- Time of exposition: 20 sec. Use functional watch.
- · After use switch off the lamp.

### SERVICE, CLEANING:

- No service necessary. Keep lamp dry and clean, particularly the lense.
- For cleaning use a wet wipe without any fusel.
  Do not use any aggressive cleaners.

#### **CHANGE OF BATTERIES:**

# (ATTENTION: DO NOT UNSCREW THE LAMP HEAD)

- Open battery case by turning lid anti clock wise at the end of the lamp and put in the two mignon-batteries (AA, Lr6 or high quality rechargeable battery e.g. Annsmann or Sanyo Eneloop). Positive pole at first.
- . Close battery case by turning lid clock wise.
- Only use faultless batteries. Leaked or damaged batteries could cause chemical burns, in that

case please use safety gloves.

 Please be sure that batteries will not be bypassed, charged or get in touch with fire. Danger of explosion!

#### DISPOSAL:

- For disposal return gadget to suitable recycling centre. Do not dispose together with household waste.
- Never dispose noxious substances like batteries together with household waste. Please return to suitable collecting points.

#### TECHNICAL DATA:

- Power supply: max. 2 AA-Batteries (each 1,5 V), recommended LR6 (Alkaline)
- LED voltage: ca. 3,3V
- LED current: max. 700mA
- . Light power: max. 500 mW
- Wave-lenght: 455-465nm
- Durability of LED: min. 1000h
- . Dimension of box: Ø26/19 x 167mm
- Weight: approximately 120g
- Operating temperature: +10°C +35°C
- Temperature for storage / transport: -10°C - +40°C
- Humidity: 35 95% rel. humidity
- Air pressure: 500 1060 hPa

# **EG- DECLARATION OF CONFORMITY:**

This gadget achieves all requirements of EMV6 / guideline for electromagnetic compatibility (2004/108/EG).

Conformity with guideline is approved by sign CE. Download of the complete declaration of conformity at www.unguisan.com.

We could also send you the complete declaration of conformity if you want.

#### WARRANTY:

- One year warranty buying this product with regular use. Except wear parts, for instance batteries! Please consider standard business conditions!
- Please use gadget only for described functions.
  Other use could damage the product or environment of the product.
- Convert or change of the product affects product safety. Warning! Risk of injury!
- Never open lamp, besides for change of battery (see above).
- Never attempt to repair item in case of malfunction!
- Take care of the product. It can be damaged caused by crushes, strokes or falls even from low height.
- Protect product from humidity and dryness.
  Never dip into liquids!
- Warning: Manufacturer is not liable for damage

of people or property, caused by wrong use! In that case the warranty will be cancelled immediately. Technical changes and errors excepted!

# NAIL CORRECTION

# Further Unguisan Products:

- Unguisan Classic Nail Prosthetics
- Unguisan Nailcare Nail Care product line

Art.-Nr. / Order No. PG-3101

### Made in Germany

Greppmayr GmbH Hainbuchenring 4 D-82061 Neuried